**FILT'RAY 2G - STERILE DISPOSABLE WATER FILTERS**

**Life Duration:** 1, 2, 3 & 4-Month

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## 1 REFERENCES

- FILT'RAY<sup>2G</sup> Shower Head Water Filter: AT030711, AT030713, AT030714.
- FILT'RAY<sup>2G</sup> Wall-Mounted Shower Water Filter: AT030777, AT030778, AT030779.
- FILT'RAY<sup>2G</sup> Faucet Water Filter with straight flow outlet: AT030717, AT030719, AT030747.
- FILT'RAY<sup>2G</sup> Faucet Water Filter with shower flow outlet: AT030718, AT030720, AT030748.

To guarantee the optimal performance of the FILT'RAY<sup>2G</sup> Disposable Water Filters, please ensure that you follow the instructions for installation (see 5) and read the warnings carefully (see 6).

## 2 GENERAL SPECIFICATIONS

- Disposable Sterile Water Filters.
- Effective against Bacteria including Opportunistic Waterborne Pathogenic Microorganisms.
- For up to 1, 2, 3 & 4 Months

FILT'RAY<sup>2G</sup> Filters are **Class I medical devices in Europe and Canada, Class II in the USA.**

## 3 INDICATIONS FOR USE

The FILT'RAY<sup>2G</sup> Disposable Water Filter is intended to be used to filter EPA (Environmental Protection Agency in USA) Quality Drinking Water. This microfiltration technology is effective for the reduction of **Healthcare-Associated Infections** by Opportunistic Waterborne Pathogenic Microorganisms. By retaining bacteria, the filters may aid in Infection Control. The filters produce water that is suitable for shower of high risk-patients, washing neonates, drinking water and ice for immunosuppressed patients, superficial wound cleansing (minor cuts, scrapes, or abrasions), cleaning of equipment used in medical procedures and washing of surgeon's hands. **The filters are not intended to provide water that can be used as a substitute for USP grade sterile water.**

## 4 DESCRIPTION OF DISPOSABLE WATER FILTER

The FILT'RAY<sup>2G</sup> Filters are intended to produce Bacteriologically Controlled Water, at Point of Use. These devices retain bacteria including opportunistic waterborne pathogenic microorganisms such as *Legionella spp.*, *Legionella pneumophila*, *Pseudomonas aeruginosa* and others via a 0.1µm nominal /0.2 µm absolute Hollow Fiber Microfiltration Membrane. This microfiltration technology ensures a higher bacteriological quality than the Potable Water of the water plumbing system. Of note, the subject FILT'RAY<sup>2G</sup> devices do not contain latex, nor tissues or by-product animal origins, nor derived of blood or any substance considered as a medicine.

FILT'RAY<sup>2G</sup> disposable water filters are designed to ensure water safety at points of use. They are recommended for managing biological risk.

**1,2, 3 & 4-month FILT'RAY<sup>2G</sup> Disposable Water Filters** provide a barrier against bacteria including opportunistic waterborne pathogenic microorganisms (*Pseudomonas aeruginosa*, *Legionella pneumophila*, etc.). These filters are designed to help control environmental conditions and are used to protect users against waterborne pathogens.

Laboratory-based microbiological challenges of *Brevundimonas diminuta*, *Legionella pneumophila* and *Pseudomonas aeruginosa* on the FILT'RAY<sup>2G</sup> demonstrate a **7-log retention capacity (see 9-2)** for all waterborne microorganisms larger than 0.1µm nominal /0.2 µm absolute and qualify the filter as sterilizing grade<sup>2</sup> filter. (see 9-B).

## 5 INSTALLATION INSTRUCTIONS

### Warnings:

- Filter installation must conform with professional aseptic techniques. Hands must be disinfected prior to installation. Inappropriate installation may result in a source of contamination or retrograde contamination.
- Make sure that the individual unit packaging (plastic bag) is not punctured or torn and that the filter is not damaged. If this is the case, do not use the device, as the performance can no longer be ensured.

### General Information:

- The shower head filter (screw-on) features a standard fitting and can therefore be connected to all flexible shower hoses.
- The faucet and wall-mounted shower filters can be connected to most plumbing fixtures using an adaptor (clip-on); contact us for a list of commercially available adaptors.
- Installation of the inline filter requires the use of rapid connectors and/or connector nozzles, contact us for a list of commercially available adaptors.
- Recommended Storage : Between 5 and 35°C (41 and 95°F)

### Installation:

1. Turn off the water supply.
2. If first installation: remove the original shower head or fit the adaptor fitting if necessary, for the faucet filter, wall mounted or in-line filter (see removal below).
3. Remove the used FILT'RAY<sup>2G</sup> Filter.
4. Carefully open the individual unit packaging (bag), making sure that the filter remains inside the bag. **Do not touch the colored end of the tap or the shower head with your fingers.**
5. **While keeping the filter in the bag**, screw the shower head filter onto the end of the hose or clip on the faucet or wall-mounted filter. Clip the inlet of the inline filter to the adaptor in the direction of the arrow indicating water flow on the label. The bag can then be removed.
7. Turn the water on for a few seconds to check that the adaptor(s) and the filter are **perfectly watertight**. In the case of leaks, repeat the installation procedure, using a new filter, if needed.

### Removal:

1. Turn off the water supply.
2. Remove the used shower head filter by unscrewing it from the end of the hose or unclipping the faucet, wall-mounted and inline filters from the adaptor.
3. Dispose of the used filter according to your institution's established procedures.

A non-leaching bacteriostatic agent is embedded into the housing material of the FILT'RAY<sup>2G</sup> disposable water filters in order to minimize the risk of retrograde contamination of the devices (see ③).

However, use in a non-controlled environment can be a source of retro-contamination.

Any filter that is believed to be contaminated must be replaced, especially if the contaminant is visible.

To do so, filters must **not** be disconnected or soaked in a cleansing / disinfectant solution.

For Australia, the product should be installed according to the current version of the Plumbing Code of Australia and AS/NZS 3500.1:2018.

### Waste management:

Used FILT'RAY<sup>2G</sup> Disposable Water Filters do not present any special infectious risk. They are classified as normal household waste. Nevertheless, we recommend that all non-recyclable plastic products be disposed of responsibly via ecological collection and treatment facilities. The installation guide and the white carton can be recycled (paper/card) (see ⑨-1).

### Maintenance:

To facilitate the tracking of maintenance procedures, a removable adhesive label (located on the filtration device) can be used to record important information for your safety records (date of installation, date of removal, filter unique identification number, etc.). This information is also in shape of UDI (Unique Device Identification) which is a datamatrix with information such as (01) GTIN: Global Trade Item Number, (10) Lot number, (17) expiration date, (21) Unit Number, (240) aqua-tools Part Number.

Manufacturing date, Expiration date and Lot number are also clearly identified on device labels and packaging.

The device label provides an additional space for the user to note clearly the date of first use and the date of disposal.

Use-life is identified by a numerical symbol featuring the number of months (1, 2, 3, 4) of the use-life specification.

## 6 WARNINGS AND CONTRAINDICATIONS

- Disposable Water Filters must not under any circumstances be used to produce water for **pharmaceutical purposes** (i.e., water that will be used in solutions intended to be administered orally, injected, or perfused).
- Disposable Water Filters **are not intended to provide water that can be used as a substitute for USP grade sterile water**.
- Disposable Water Filters are single-use (disposable). They cannot be sterilized or re-installed on another point of use.
- Disposable Water Filters have a **limited time use** and should not be used once **expiration date** (indicated on the label) is **past due**.
- Disposable Water Filters must not come into direct contact with germ sources (dirty skin or hands, feces...).
- Disposable Water Filters must not be struck or dropped (fall). Filter performance is not guaranteed if the filter casing is physically damaged. Do not attempt to open the device.
- Disposable showerhead filters should always be placed on the wall bracket after use. They must not be unscrewed at any point during the period of use. The same applies for the faucet, the wall-mounted shower and the inline filters which must not be unclipped at any time during the period of use except for when the water supply network is disinfected (at which time a new filter must be installed).

## 7 TECHNICAL SPECIFICATIONS

### Conditions for use:

The performance of FILT'RAY<sup>2G</sup> Disposable Water Filters is guaranteed when the water pressure and temperature remain between 14.5 and 72.5 PSI and 41°F and 140°F, respectively. Flow rates measured under normal conditions are as follows:

Product	Typical Flow Rate (gal/min)				
	Water Pressure (PSI)				
	14.5	29	43.5	58	72.5
FILT'RAY <sup>2G</sup> Shower Head Filter	1.2	2.3	3.1	3.9	4.5
FILT'RAY <sup>2G</sup> Wall-Mounted Shower Filter	1.0	2.4	3.4	4.5	5.1
FILT'RAY <sup>2G</sup> Faucet and In-Line Filter	0.6	1.2	1.7	2.0	2.3

\* Flow rate measurements are determined with the filters installed on a faucet.

Product	Typical Flow Rate (L/min)				
	Water Pressure (Bar)				
	1	2	3	4	5
FILT'RAY <sup>2G</sup> Shower Head Filter	4.5	8.7	11.9	14.9	16.9
FILT'RAY <sup>2G</sup> Wall-Mounted Shower Filter	3.9	8.9	13	16.9	19.4
FILT'RAY <sup>2G</sup> Faucet and In-Line Filter	2.2	4.7	6.3	7.7	8.8

\* Flow rate measurements are determined with the filters installed on a faucet.

### Heat and chemical resistance:

Disposable Water Filters are resistant to the thermal and chemical shocks that frequently occur during disinfection of the water supply system (see 9-1). They remain effective after treatment at **70°C (158°F) for 30 min** (cumulative maximum) or a dose of active chlorine at **100 ppm for 60 min** (cumulative monthly maximum: FILT'RAY<sup>2G</sup> 1-Month Filters: 1 hour at 100ppm, FILT'RAY<sup>2G</sup> 2-Month Filters: 2 hours at 100ppm, FILT'RAY<sup>2G</sup> 3-Month Filters: 3 hours at 100ppm and FILT'RAY<sup>2G</sup> 4-Month Filters: 4 hours at 100ppm).

After exposure to such treatments, please note that there may be a reduction in water flow due to contamination of the water supply with various types of particles. In such cases, the water filter should be replaced with a new filter should the flow rate become greatly reduced and/or unacceptable.

### Blockage tolerance:

FILT'RAY<sup>2G</sup> Disposable Water Filters have a certain amount of tolerance to blockage with the **Bubl'AirWash Technology** (see 9-A), which is a mechanism for extending the Filter's life. Nevertheless, the flow rate at the point of use may decrease somewhat before the end of lifetime if there are unusually high levels of particles or suspended microorganisms in the water supply (for example, after a maintenance procedure has been performed). If the flow rate becomes too slow for user comfort, simply replace the filter.

## 8 PRODUCT COMPLIANCE

The Sterile FILT'RAY<sup>2G</sup> Disposable Water Filters are sterilized by gamma irradiation in compliance with ISO 11137.

### Do not re-sterilize.

Sterile FILT'RAY<sup>2G</sup> Disposable Water Filters:

- Are **Class II Medical Devices in the US** - FDA-510K Clearance
- Are **Class I Medical Devices for Europe** that comply with directive 93/42/CEE. **CE marking 0459: year 2020.**
- Are **Class I Medical Devices for Canada** that comply with **SOR/98-282.**
- A non-leaching bacteriostatic agent is embedded into the housing material FILT'RAY<sup>2G</sup> Disposable Water Filters in order to minimize the risk of retrograde contamination of the devices. Laboratory based microbiological testing has demonstrated that the bacteriostatic agent reduces external surface contamination by more than 99.9% for *Pseudomonas aeruginosa*, *Escherichia coli*, *Staphylococcus aureus* and *Methicillin Resistant Staphylococcus aureus* (MRSA) (see 9-2).
- Comply with Drinking Water Certifications (**ACS** - France, **KTW** - Germany, **WRAS** - United Kingdom, **WaterMark** - Australia/ New Zealand).
- Meet the requirement of the Australian/New Zealand Standards **AS/NZS 3497** "Drinking water treatment units" and **AS/NZS 4348** "Water supply - Domestic type water treatment appliances - Performance requirements" for the class II(a): microbiological treatment - bacteria removal. Certification and **WaterMark** marking. For more information, please refer to the performance data sheet – available upon request.

## 9 MANUFACTURING INFORMATION

FILT'RAY<sup>2G</sup> Disposable Water Filters are manufactured in France

By: **aqua-tools** | 26, rue Charles-Édouard Jeanneret | 78300 POISSY | France

- For more information, visit our website at: [www.aqua-tools.com](http://www.aqua-tools.com)  
or contact us: **Phone:** + 33 1.39.75.02.20. **E-mail :** [contact@aqua-tools.com](mailto:contact@aqua-tools.com)
- The "Scientific and Technical Validation Guide" for the relevant product is available upon request.

### Reference:

- A.** The Bub'Air Wash™ Technology is a mechanism to extend the life of the Filter which introduces turbulent flow during filtration to keep particulates from becoming fully trapped on the membrane and permits the filters to have long-life use.
- B.** Sterilizing grade filter is defined in "American standard Test Method ASTM F838-15a and in FDA - Guidance for Industry - Sterile Drug Products Produced by Aseptic Processing - Current Good Manufacturing Practice" as a filter ability to sustain a bacterial challenge of 10<sup>7</sup> CFU of *Brevundimonas diminuta* per cm<sup>2</sup> of filtration surface.

### Product label - meaning of symbols

(Rx only: "Caution: Federal Law restricts this device to sale by or on the order of a licensed practitioner.")

<b>REF</b>	<b>LOT</b>	
Catalog number	Batch code	Use by
<b>STERILE R</b>		
Sterilized using irradiation	Do not resterilise	Do not use if packaging is damaged
		
Free from natural latex	Manufacturer	Consult instructions for use
<b>CE</b> 0459	 CMA-WN-080040 AS/NZS 3497	
Compliant with the essential requirements - Sterile	WaterMark marking: License number for AS/NZS 3497 compliance by CertMark International	

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## 1 REFERENCES

- FILT'RAY<sup>2G</sup> Shower head water filters: AT030711, AT030713, AT030714, AT030716.
- FILT'RAY<sup>2G</sup> Wall-mounted shower water filters: AT030777, AT030778, AT030779, AT030780.
- FILT'RAY<sup>2G</sup> Tap water filters with straight flow outlet: AT030717, AT030719, AT030747, AT030749.
- FILT'RAY<sup>2G</sup> Tap water filters with shower flow outlet: AT030718, AT030720, AT030748, AT030750.
- FILT'RAY<sup>2G</sup> In-line water filters: AT030733, AT030734, AT030745, AT030746.

To guarantee the optimal performance of the FILT'RAY<sup>2G</sup> disposable water filters, please ensure that you follow the instructions for installation (see 5) and read the warnings carefully (see 6).

## 2 GENERAL SPECIFICATIONS

- Disposable sterile water filters.
- Effective against bacteria including opportunistic waterborne pathogenic microorganisms.
- 1, 2, 3 & 4 months

FILT'RAY<sup>2G</sup> filters are **class I medical devices** in Europe and Canada, classe II in the USA.

## 3 INDICATIONS FOR USE

FILT'RAY<sup>2G</sup> 1, 2, 3 & 4-Month sterile water filters provide a barrier against waterborne microorganisms such as bacteria (*Legionella pneumophila*, *Pseudomonas aeruginosa* and other Gram negative bacilli, nontuberculous Mycobacteria), fungi (*Aspergillus sp.*, *Fusarium sp.*), protozoa and particles originating in the water distribution system. The microfiltration membrane protects **immune-suppressed patients** and vulnerable people. Filters are necessary for the care of hospitalised, **immune-suppressed patients** in high-risk areas such as bone marrow transplant and organ transplant units, oncology services, intensive care units, resuscitation, neonatal services and any other places that care for this type of patient.

## 4 DESCRIPTION

The FILT'RAY<sup>2G</sup> filters are intended to produce bacteriologically controlled water at the point-of-use. These devices retain bacteria, including opportunistic waterborne pathogenic microorganisms such as *Legionella spp.*, *Legionella pneumophila*, *Pseudomonas aeruginosa* and others via a microfiltration membrane. This microfiltration technology ensures as a higher bacteriological quality than the potable water of the water plumbing system. Please note, the FILT'RAY<sup>2G</sup> devices do not contain latex, nor tissue or by-products of animal origin, neither are they derived from blood or any substance considered as a medicine.

FILT'RAY<sup>2G</sup> disposable water filters are designed to ensure water safety at the point-of-use. They are recommended in the procedures for **management and prevention of healthcare associated infections**.

FILT'RAY<sup>2G</sup> filters use a 0.1µm nominal and 0.2µm absolute tubular membrane filtration technology of sterilising grade (see 9-A). The filters provide a **high-performance level of water microfiltration** while delivering a **comfortable flow** for users.

Microbiological challenges, conducted in laboratories with *Brevundimonas diminuta*, *Legionella pneumophila* and *Pseudomonas aeruginosa* species on FILT'RAY<sup>2G</sup> filters, demonstrate a **retention efficiency of 7 log** (see 9-2) for all water-borne microorganisms greater than 0.1µm nominal / 0.2µm absolute and qualify the filter as a sterilising grade filter (see 9-A).

## 5 INSTALLATION INSTRUCTIONS

### Warnings:

- Filter installation must conform with professional aseptic techniques. Hands must be disinfected prior to installation. Inappropriate installation may result in a source of contamination or retro-contamination.
- Make sure that the individual unit packaging (plastic bag) is not punctured or torn and that the filter is not damaged. If this is the case, do not use the device, as the performance can no longer be ensured.

### General information:

- The shower head filter (screw-on) features a standard fitting and can therefore be connected to all flexible shower hoses.
- The tap, wall-mounted shower and in-line filters require fast-fix connectors and/or tubing connectors, contact us for a list of commercially available adaptors.

### Installation:

1. Turn off the water supply.
2. If this is the first installation: remove the original shower head or fit the adaptor, if necessary, for the tap, wall-mounted shower or in-line filter (see removal below).
3. Remove the used FILT'RAY<sup>2G</sup> filter.
4. Carefully open the individual unit packaging (bag), making sure that the filter remains inside the bag. **Do not touch the coloured end of the tap or the shower head with your fingers to avoid contamination.**
5. **Keeping the filter in the bag**, screw the shower head filter onto the end of the hose or clip on the tap or wall-mounted shower filter. Clip the inlet of the in-line filter on to the adaptor in the direction of the arrow indicating water flow on the label.
6. The bag can then be removed.
7. Turn the water on for a few seconds to check that the adaptor(s) and the filter are **perfectly watertight**. In the case of leaks, repeat the installation procedure, using a new filter if needed.

## Removal:

1. Turn off the water supply.
2. Remove the used shower head filter by unscrewing it from the end of the hose or unclipping the tap, wall-mounted shower and/or inline filter from the adaptor.
3. Dispose of the used filter according to your institution's established procedures.

A non-leaching bacteriostatic agent is embedded into the housing material of the FILT'RAY<sup>2G</sup> disposable water filters in order to minimise the risk of retro-contamination of the devices (see 6).

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## Waste management:

Used filters do not present any special infectious risk. They are classified as **normal household waste**. Nevertheless, we recommend that all non-recyclable plastic products be disposed of responsibly via ecological collection and treatment facilities (see 9-1). The installation guide and the white carton can be recycled (paper/card).

## Maintenance:

To facilitate the tracking of maintenance procedures, a removable adhesive label (located on the filtration device) can be used to record important information for your safety records (date of manufacture, date of removal, batch number and filter unique identification number, etc.). Stick on to the tracking page and write down the installation and removal date. This information is also available as the UDI (Unique Device Identification) which is a data matrix with information in the form of (01) GTIN: Global Trade Item Number, (10) Lot number, (17) expiration date, (21) Unit Number, (240) aqua-tools part number.

Manufacturing date, Expiration date and Lot number are also clearly identified on device labels and packaging. The lifespan is identified by a numerical symbol featuring the number of months.

## 6 CONTRAINDICATIONS

FILT'RAY<sup>2G</sup> Disposable Water Filters:

- Must not, under any circumstances, be used to produce water for **pharmaceutical purposes** (i.e., water that will be used in solutions intended to be administered orally, injected, or perfused).
- Are single use (disposable) and cannot be re-sterilised or re-installed on another point-of-use.
- Have a **limited lifespan** according to the filter used, and an **expiration date** indicated on the label. These must be adhered to.
- Must not come into direct contact with germ sources (dirty skin or hands, contaminated aerosols, body fluids, etc.).
- Must not be unscrewed or disconnected during the life of the filter except if the water distribution system is disinfected. In this instance, replace with a new filter.
- Must not be struck or dropped (fall). Filter performance is not guaranteed if the filter casing is physically damaged. Do not attempt to open the device.
- Shower head filters should always be placed on the wall bracket after use. They must not be unscrewed at any point during the period of use. The same applies for the tap, the wall-mounted shower and the inline filters which must not be unclipped at any time during the period of use except for when the water supply network is disinfected (at which time a new filter must be installed).

## 7 TECHNICAL SPECIFICATIONS

### Conditions for use:

The performance of FILT'RAY<sup>2G</sup> disposable water filters is guaranteed when the water pressure and temperature remain between 1 to 5 bar and 5°C and 60°C, respectively. Flow rates measured under normal conditions are as follows:

		FILT'RAY <sup>2G</sup> Shower Head Filter	FILT'RAY <sup>2G</sup> Tap and In-line Filter*
		Flow rate (l/min)	
Water pressure (bar) static or dynamic	1	4.5	2.2
	2	8.7	4.6
	3	11.9	6.3
	4	14.9	7.7
	5	16.9	8.8

\* tap filters with low water consumption

### Heat and chemical resistance:

FILT'RAY<sup>2G</sup> disposable water filters are resistant to the thermal and chemical shocks that frequently occur during disinfection of the water supply system (see 9-1). They remain effective after treatment at **70°C for 30 min** (cumulative maximum) or a dose of active chlorine at **100 ppm for 60 min** (cumulative monthly maximum): FILT'RAY<sup>2G</sup> 1-month filters: 1 hour at 100ppm; 2-month filters: 2 hours at 100ppm; 3-month filters: 3 hours at 100ppm; and 4-month filters: 4 hours at 100ppm.

### Blockage tolerance:

FILT'RAY<sup>2G</sup> Disposable Water Filters have a certain amount of tolerance to blockage with the **Bubl'AirWash Technology** (see 9-B). Nevertheless, the flow rate at the point of use may decrease somewhat before the end expiration date if there are unusually high levels of particles or suspended microorganisms in the water supply (for example, after a maintenance procedure has been performed). If the flow rate becomes too slow for user comfort, simply replace the filter.

## 8 PRODUCT COMPLIANCE

Sterile FILT'RAY<sup>2G</sup> Disposable Water Filters:

- Are sterilised by gamma irradiation in compliance with ISO 11137. **Do not re-sterilise.**
- Are **Class I Medical Devices in France** - complying with directive 93/42CEE. **CE marking 0459: year 2020.**
- Are **Class II Medical Devices in the USA (FDA-510K clearance).**
- Dispose of the used filter according to your institution's established procedures. **Class I Medical Devices in Canada** complying with DORS/98-282.
- A non-leaching **bacteriostatic agent** is embedded into the housing material to minimise the risk of retro-contamination of the devices. Laboratory based microbiological tests have demonstrated that the bacteriostatic agent allows a reduction in the external surface contamination by more than 99.9% for *Pseudomonas aeruginosa*, *Escherichia coli*, *Staphylococcus aureus* and *Methicillin-resistant staphylococcus aureus* (MRSA) (see 9-2).
- Comply with drinking water certifications (**ACS-France**, **KTW-Germany**, **WRAS-United Kingdom**, **WaterMark-Australia/New Zealand**).
- Meet the requirements of the Australian/New Zealand Standards **AS/NZS 3497** "Drinking water treatment units" and **AS/NZS 4348** "Water supply - Domestic type water treatment appliances - Performance requirements" for the class II(a): microbiological treatment - bacteria removal. Certification and **WaterMark** marking. For more information, please refer to the performance data sheet - available upon request.

## 9 MANUFACTURING INFORMATION

FILT'RAY<sup>2G</sup> Disposable Water Filters are manufactured in France by:

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2. The "Scientific and Technical Validation Guide" for the relevant product is available on request.

### References:

- A.** Sterilising grade filter is defined in "American standard Test Method ASTM F838-15a and in FDA - Guidance for Industry - Sterile Drug Products Produced by Aseptic Processing - Current Good Manufacturing Practice" as a filter ability to sustain a bacterial challenge of  $10^7$  CFU of *Brevundimonas diminuta* per  $cm^2$  of filtration surface.
- B.** The Bubl'Air Wash™ Technology is a mechanism to extend the service life of the filter which introduces turbulent flow during filtration to keep particulates from becoming fully trapped on the membrane and permits the filters to have long-life use.

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		
Catalogue number	Batch code	Use by
		
Sterilised using irradiation	Do not re-sterilise	Do not use if packaging is damaged
		
Free from natural latex	Manufacturer	WaterMark marking: License number for AS/NZS 3497 compliance by CertMark International
		
Non-sterile	Consult instructions for use	Compliant with the essential requirements

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